wellion®

uet

AT

DE

WellionVet® GLUCO CALEA

Teststreifen

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EN WellionVet® GLUCO CALEA Test strips

IMPORTANT: PLEASE READ THIS INFORMATION AND YOUR WELLIONVET GLUCO CALEA USER GUIDE BEFORE USING THE GLUCO CALEA TEST STRIPS TO TEST YOUR BLOOD SUGAR.

INTENDED USE
The WellionVet® GLUCO CALEA Blood Glucose Test Strips are to be used with the WellionVet® GLUCO CALEA Blood Glucose Meters to quantitatively measure glucose in capillary whole blood.

Warnings and Precautions
[1] Do not use test strips after expiration date. The expiration date is printed on the carton and on the label of the test strip bottle. [2] Do not use test strips that are wet or damaged. [3] Do not re-use the strips. [4] Use the WellionVet® GLUCO CALEA Blood Glucose Monitoring System within temperature range of 10°C to 40°C (50°F to 104°F).

Storage and Handling
[1] Always close the vial cap tightly after removing a test strip to avoid moisture and direct sunlight. [2] Store the test strips at temperatures between 4°C to 30°C (39°F to 86°F). [3] Store out of direct sunlight; do not refrigerate or freeze. [4] Unopened test strips are stable until the expiration date printed on the bottle when stored properly. [5] The month printed refers to the END of that month. [6] Use within 3 months after first opening. [7] Do not handle the test strips with wet or dirty hands.

Test Procedure
Refer to "Testing Your Blood Glucose" section in the User's Guide of the WellionVet® GLUCO CALEA Blood Glucose Meter.

Problem Solving
[1] Confirm the Test Strips are within the expiration date. [2] Make sure the blood completely fills the reaction zone. An error will be displayed on the WellionVet® GLUCO CALEA meter if there is insufficient blood. DO NOT add a second drop of blood; discard the test strip from the meter and retest using a new Test Strip.

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